# Lifeloc

TECHNOLOGIES

## FC5 HORNET OPERATIONS MANUAL

## Table of Contents

|                             |        |
|-----------------------------|--------|
| Components Diagram .....    | 3      |
| Battery Installation .....  | 4      |
| Test Modes Explained .....  | 4 - 5  |
| Operations.....             | 5 - 6  |
| User Prompts .....          | 7      |
| Specifications.....         | 8      |
| Calibration.....            | 8      |
| Service & Maintenance ..... | 8      |
| Spanish.....                | 9 - 13 |

The FC5 Hornet is a new generation of alcohol screener for use by schools, police, probation, jails, mining and related testing in safety sensitive industries.

Testing for the presence of alcohol has never been easier. No mouthpieces or special attachments are required. Simple to operate, the FC5 uses icons to eliminate misinterpretation due to language barriers.

The FC5 is alcohol specific and uses the same 1 inch platinum fuel cell used in Lifeloc's law enforcement breathalyzers providing users with larger breath sample size, more rapid response and reset times, and long working life.

The FC5 Hornet comes standard with three operating test modes used for subject, ambient air and open container testing. Designed for safety, the FC5 allows the operator to see the test results without revealing the display to the subject.

## Components Diagram

- 1. LCD Display:** Displays test order and results with an automatic backlight for day/night visibility
- 2. Function Button:** Scrolls through test modes
- 3. Power Button:** Turns the unit on or off. Unit will automatically power off after 5 minutes of inactivity
- 4. Execute Button:** Allows you to perform functions or trigger a manual override test
- 5. Sampling Cone:** Breath should be directed to the cone in order to ensure a proper test
- 6. Protective Grip:** Covers and protects the entire unit and any exposed ports
- 7. Battery Door:** Holds 4 AA batteries that allow for 4700 tests

## Battery Installation

The FC5 is designed to operate using 4 AA alkaline batteries. Remove the battery door by pressing in and down on the door located on the lower back of the FC5.

Install the 4 AA alkaline batteries in the direction of the symbols in the battery case. Close the case by pushing up on the battery door until it locks shut.

In order to change or install batteries, the grip must be removed from the FC5.

## Test Modes Explained

### POS+/- NEG Test Mode

Perfect for zero tolerance environments, this test mode displays results as **POS +** or **- NEG** for alcohol.

A **POS +** test is any alcohol level  $> .000$  BAC.

Tests are triggered automatically once the subject exhales enough breath towards the collector cone.

Manual override is available using the execute button for testing uncooperative subjects or discrete ambient air.

### LEVELS Test Mode

Used for more precise alcohol testing, levels test mode, will alert the operator to levels of alcohol in the subject. Equipped with automatic testing with manual override capabilities for difficult subjects.

### Pre-Set Alcohol Levels

Zero:  $< .010\%$  BAC ( $< .04$  mg/l)

Low:  $.011 - .020\%$  BAC ( $.05 - .09$  mg/l)

High:  $> .021\%$  BAC ( $> .100$  mg/l)

## OPEN CONTAINER Test Mode

Samples air-space over open drink containers or spills. With its unique double pump action, the Open Container mode ensures sufficient test sample even from small neck bottles. One button sample triggering allows for precise positioning over the sample and the quiet pump action ensures discrete and unobtrusive sampling.

## Operations

### Getting Started

1. Turn on the FC5 by momentarily pressing the **Power** button. The instrument will initiate a self-test and then display the POS/NEG test mode.
2. To switch test modes, press the **Function** button and scroll through your options.
3. Once a test mode is selected, it will become the new default mode. You can change defaults by momentarily pressing the **Power** button, switching modes using the **Function** button or by completely powering the unit off.
4. You are now ready to test.

### Conducting an Automatic Test

1. Position the sampling cone about 4" or 10 cm from the subject's mouth.
2. Instruct the subject to blow steadily towards the cone.
3. The test will automatically trigger once the subject has provided an adequate sample.
4. Read results.
5. Results will stay on the screen for 30 seconds or until you press the **Function** button.

### Conducting a Manual Test

1. Position the sampling cone no less than 4" from the subject's mouth.
2. Instruct the subject to blow steadily towards the cone.
3. Once the subject begins blowing, press the **Execute** button to trigger the test.
4. Read results.
5. Results will stay on the screen for 30 seconds or until you press the **Function** button.

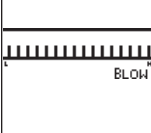
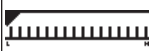
### Conducting an Open Container Test

1. Position the sampling cone no less than 4" from the container or sample.
2. Press the **Execute** button to trigger the test.
3. Read results.
4. Results will stay on the screen for 30 seconds or until you press the **Function** button.
5. For best results, slightly agitate the sample prior to trigger the test.

### Conducting an Ambient Air Test

1. In any of the three modes, wave the unit in the air and press the **Execute** button to trigger the test.
2. Read results.
3. Results will stay on the screen for 30 seconds or until you press the **Function** button.

## User Prompts

| User Prompts  | Cause & Course of Action   |
|---|--|
| POS+/- NEG  | POS/NEG Test Mode: NEG results will register as a Zero, POS results will be anything >0.   |
| - NEG   | Negative results when in POS/NEG Test Mode.  |
| POS +   | Positive result when in POS/NEG Test Mode.   |
|  | Levels Test Mode: Results will register in one of three different levels depending on the subjects alcohol content.<br>Zero: <.010% BAC (<.04 mg/l)<br>Low: .011 - .020% BAC (.05 - .09 mg/l)<br>High: >.021% BAC (>.100 mg/l) |
|  | Display shows results as <.010% BAC when in Levels Test Mode.  |
|  | Display shows results as .011 - .020% BAC when in Levels Test Mode.  |
|  | Display shows results as >.021% BAC when in Levels Test Mode.  |
|  | Open Container Test Mode: NEG results will register as a Zero, POS results will be anything >0.  |
|  | Negative results when in Open Container Test Mode.   |
|  | Positive results when in Open Container Test Mode.   |
|  | The hourglass appears when the FCS is in wait mode. This can occur when the FCS is recovering from a POS test.   |

## Specifications

|                  |   |
|------------------|---|
| Sensor           | Electrochemical Fuel Cell               |
| Sampling         | Passive                                 |
| Power Supply     | 4 AA Batteries                          |
| Battery Life     | Approx. 4700 tests                      |
| Operating Temp.  | 32°F - 122°F (0°C - 50°C)               |
| Display          | High Resolution Graphic Display         |
| Size             | 1.25" x 2.5" x 5" (31.7 x 63.5 x 127mm) |
| Weight w/Battery | 5oz (142 grams)                         |
| Trigger Mode     | Automatic and Manual                    |
| Warranty         | 1 year limited warranty                 |

## Calibration

All breath testers require periodic calibration to maintain their accuracy. Your FC5 Hornet is designed to be calibrated by a factory certified calibration technician at least once every 12 months. To have your instrument calibrated or serviced email Lifeloc at [service@lifeloc.com](mailto:service@lifeloc.com), visit us at [www.lifeloc.com](http://www.lifeloc.com), call 1.800.722.4872 or contact your local distributor.

## Service & Maintenance

For general maintenance we recommend wiping the unit off with a soft cloth sprayed with a non-abrasive disinfectant cleaner. To obtain the longest life from your FC5 Hornet, it is recommended that two or three tests be taken at least every 30 days. This prevents the fuel cell from drying out over extended periods of non use. If you require service, email us at [service@lifeloc.com](mailto:service@lifeloc.com), call 1.800.722.4872 or contact your local distributor.

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